

# Microlet™ next Lancing Device



## Steps to lancing success



Ascensia Diabetes Care Holdings AG,  
Peter Merian-Strasse 90, 4052 Basel, Switzerland.

Ascensia, the Ascensia Diabetes Care logo and Contour are trademarks and/or registered trademarks of Ascensia Diabetes Care Holdings AG.

© Copyright 2018 Ascensia Diabetes Care Holdings AG. All rights reserved.



## 1 Prepare



### Twist and unlock the endcap as shown

Line up the white raised mark and black indented mark



### Loosen the round protective cap on the lancet but do not remove it

Rotate 1/4 turn



### Insert the lancet firmly into the lancing device

Push until the lancet comes to a complete stop



### Carefully twist off the round protective cap

Save the cap for disposing of the used lancet



### Re-attach the endcap

'Click' to lock

## 2 Lance



### Rotate the endcap dial to adjust the puncture depth

1 for the shallowest and 5 for the deepest



### Pull back the reset handle until the release button is raised



### Press the lancing device firmly against the side of your finger and press the release button

You will hear a click when the lancet is released

## 3 Eject



### Remove the endcap

Place the protective cap that you saved onto a flat surface. Push the lancet into the cap as shown



### Push the grey lancet ejector forward until the lancet falls into the container, then slide ejector back

Don't forget to re-attach the endcap

Before use and for full information please see the MICROLET™NEXT packaging insert<sup>1</sup>