


**Contour**  
Evolving with you

# Seamlessly share data with your healthcare professional


**CONTOUR®DIABETES App** and **GlucoContro.online**



**Add More Information** to help put your readings in context, eg medication, activities and carb intake

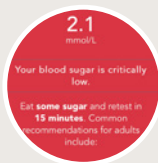

**Get Smart Alerts** when your levels are at a critical low or critical high

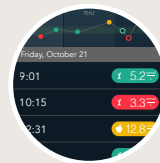

**Consistent Colour Target Range Indicators** on your meter, app and GlucoContro to understand your readings

Pair your **CONTOUR®NEXT** meter with **CONTOUR®DIABETES App**

Follow the prompts in the CONTOUR®DIABETES App by going to the **Menu** > **Settings** > **My Meters** then click **+**. Alternatively you can watch the video from our website in **Support** > **Instructional Videos**

**Download now**



**You can be confident that your data is protected:**

The **CONTOUR®DIABETES App** applies appropriate safeguards to ensure your personal data is processed securely and in compliance with applicable laws.

To see if your smartphone is compatible with this app, visit [compatibility.contourone.com](http://compatibility.contourone.com)



**Contour**  
Evolving with you

# Seamlessly share data with your healthcare professional

## CONTOUR®DIABETES App and [GlucoContro.online](#)

 **GlucoContro**  
.online



**Access notes\*** from your appointment with your healthcare professional



**Appointment notes** available when shared by your healthcare professional



**One less step before your appointment - eliminates sending a report**

Your healthcare professional can access your shared data remotely and in real time when you approve data sharing. Data available as soon as it is uploaded to the CONTOUR®CLOUD.

**Login to GlucoContro.online with the email and password (credentials) you have used for CONTOUR®DIABETES App.**

When it comes to blood glucose management, **trust CONTOUR®**   @ContourAscensia.AU

**Always read instructions for use.**  
**Consult your healthcare professional if this product is right for you.**

\*You need to be registered in GlucoContro.online


SdNcenter is the legal manufacturer and developer of GlucoContro.online and is partnering with Ascensia to market and deliver the tool to HCP and Patient users.

© Copyright 2022, Ascensia Diabetes Care Holdings AG. All rights reserved. Ascensia, the Ascensia Diabetes Care logo and Contour are trademarks and/or registered trademarks of Ascensia Diabetes Care Holdings AG. Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc. Google Play and the Google Play logo are trademarks of Google LLC. All other trademarks are properties of their respective owners and are used solely for informative purposes. No relationship or endorsement should be inferred or implied.

All other trademarks are properties of their respective owners and are used solely for informative purposes. No relationship or endorsement should be inferred or implied.


### **Email Invitation**

1. Your HCP will send you an email.
2. Click the link in the email to accept the invitation then sign in with your CONTOUR®DIABETES App credentials.
3. Once logged in, click the menu on the left to see your HCP's name and clinic. Click Accept.
4. A second confirmation box will appear, click "Yes, share data".

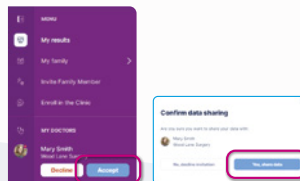

### **Using the Clinic Code**

1. Login to GlucoContro.online.
2. Open the left menu bar and click "Enrol in the Clinic".

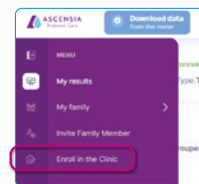

3. In the pop up box, enter the following clinic code and press the "Submit" button
- 
4. Close your confirmation box and wait for your HCP to accept your request.

  
**ASCENSIA**  
Diabetes Care